

Product Information

243103 NADAL® COVID-19 Ag Test

© This document is intellectual property of nal von minden GmbH. Utilisation or reporting its contents is forbidden, unless explicitly permitted. Violations will result in legal consequences.




Rapid Tests

Laboratory Diagnostics

Laboratory Service

Consulting & Service

Product	Sample material	Test time	Product code	Stückzahl
COVID-19 Antigen Rapid test for the detection of SARS-CoV-2 nucleoprotein				
NADAL® COVID-19 Ag test cassette	naso- or oropharyngeal swab	15 min	243103N-20	20

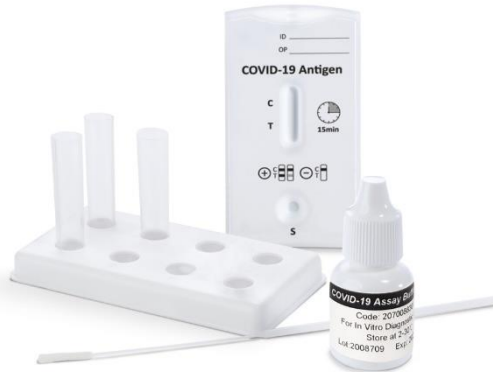


Performance

- ✓ Sensitivity: **97.56 %** (Ct range 20-30)
- ✓ Specificity: **> 99.9 %**

Sales limitations

- ✓ Professional use only

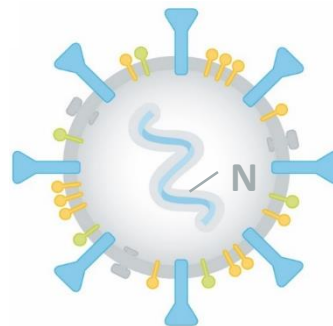


Nucleocapsid protein (N)

- Encases the viral RNA and is involved in replication, transcription and packaging of the viral genome
- Advantage: highly abundant and specific

NADAL® COVID-19 Ag Test – Sample collection

Product	Sample material	Test time	Product code	Stückzahl
COVID-19 Antigen Rapid test for the detection of SARS-CoV-2 nucleoprotein				
NADAL® COVID-19 Ag test cassette	naso- or oropharyngeal swab	15 min	243103N-20	20



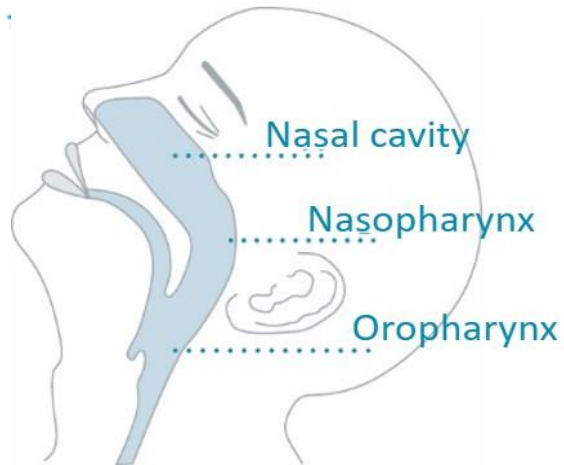


naso- or oropharyngeal swab

15 min

243103N-20

20

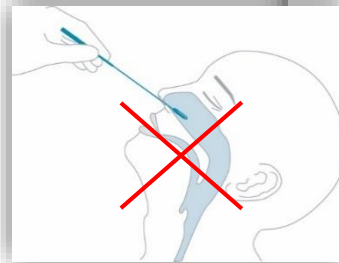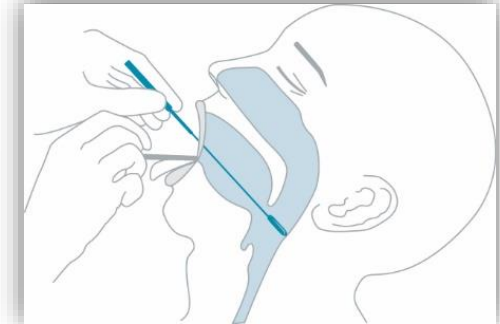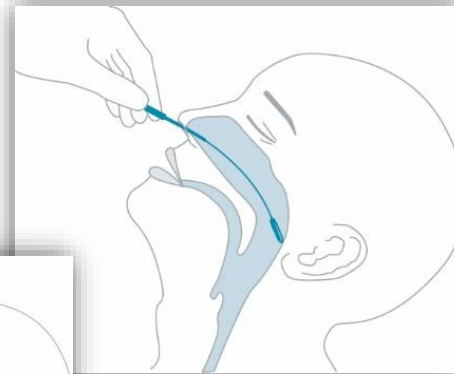

Specimen collection is a critical point – what must be taken into account?

✓ **Sample quality** (slide [3](#))

✓ **Time window**

→ On average up to 8 days after the onset of symptoms, the virus conc. in the upper respiratory tract is highest; thereafter the virus tends to migrate to the lower respiratory tract, i.e. detection by means of a throat swab is becoming increasingly less likely (van Kampen et al., Woelfel et al., Bullard et al.)

✓ **Course of infection**

→ Asymptomatic patients can carry lower viral loads (Chau et al.)

✓ **Viral transport media (VTM) without** denaturing components are compatible


- Denaturing = breaking protein structure
- VTM-N incompatible due to denaturing components, e.g. guanidine salt
- **Use of smallest possible volume (0.5 - max. 1 mL) recommended**
 - high dilutions result in a loss of sensitivity

✓ **Swab tip material**

- Only the swabs included in the test kit can guarantee flawless test performance
 - we advise against the use of swabs other than those supplied in the test kit

NADAL® COVID-19 Ag Test – Test procedure

Product	Sample material	Test time	Product code	Stückzahl
COVID-19 Antigen Rapid test for the detection of SARS-CoV-2 nucleoprotein				
NADAL® COVID-19 Ag test cassette	naso- or oropharyngeal swab	15 min	243103N-20	20

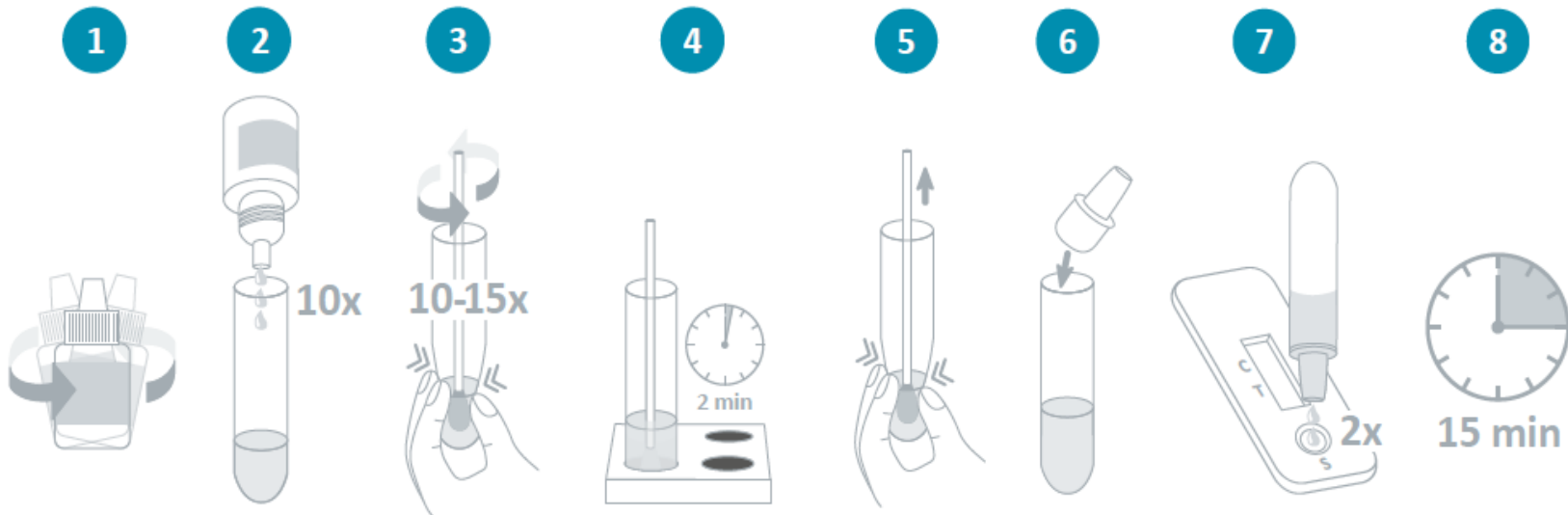

More detailed information can be found in the instructions for use.

Benefits of using the NADAL® COVID-19 Ag Test


- ✓ Affordable and easy to use – no instrumentation necessary
- ✓ All necessary test components included
- ✓ Compatible with viral transport media (VTM)
- ✓ No cross-reactivity with human pathogenic coronaviruses (like hCoV-229E, -HKU1, -NL63 and -OC43), influenza A/B viruses or other respiratory pathogens

- ✓ Fast and reliable results in just 15 minutes
- ✓ Highly sensitive with high viral loads (Ct range 20-30) (see slides [7](#) & [8](#))
- ✓ Closing a diagnostic gap with antigen testing (see slide [9](#))

Benefits of using the NADAL[®] COVID-19 Ag Test


- ✓ Highly sensitive with high viral loads (Ct range 20-30)
- ✓ comparable sensitivity to PCR at Ct values ≤ 30

Sensitivity of antigen test vs. PCR cycle threshold

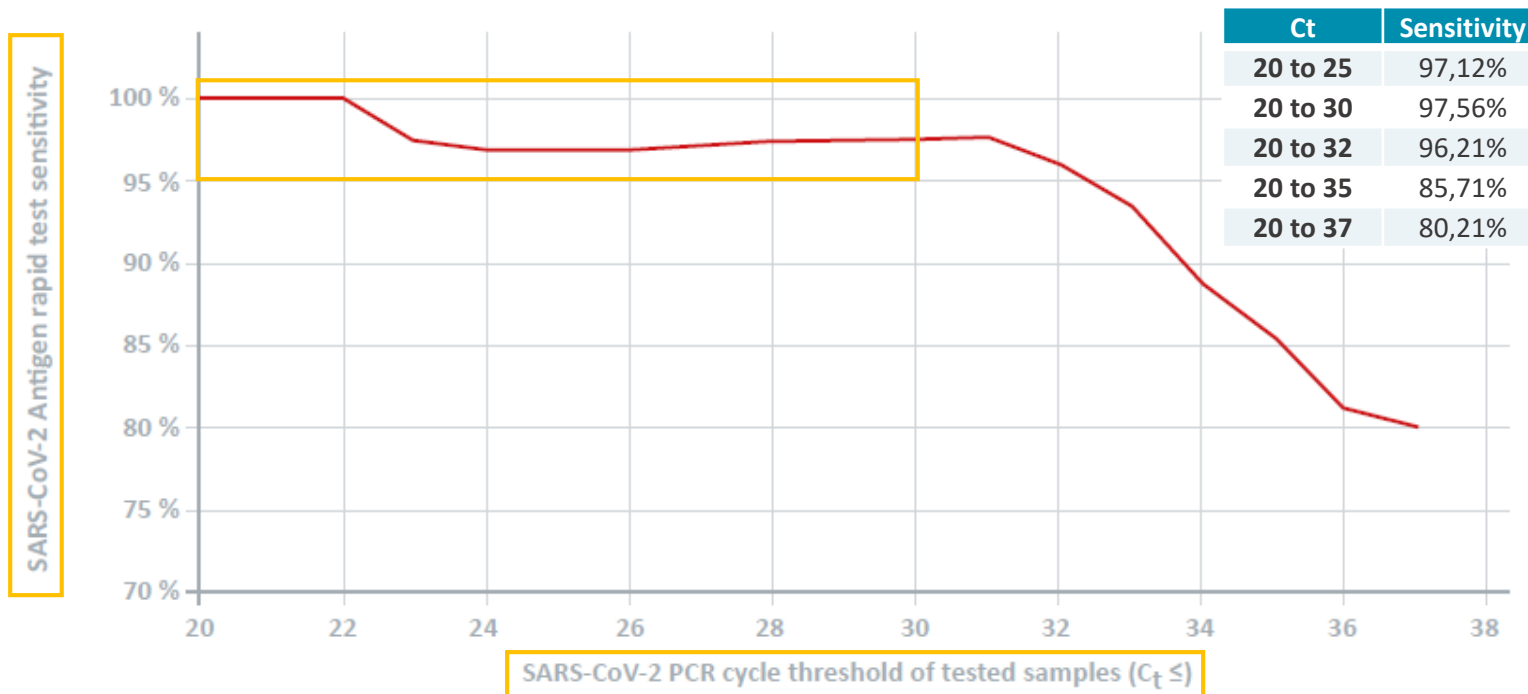


qRT-PCR (quantitative reverse transcription PCR)

✓ qPCR workflow:

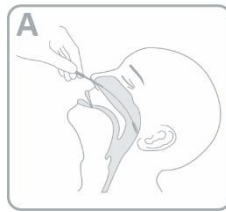


Sample collection

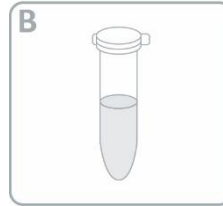

RNA Isolation



qRT-PCR

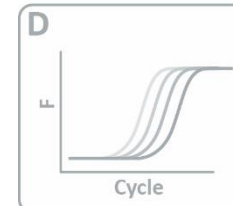


Result interpretation



✓ Cycle threshold (Ct):


- the number of cycles required for the fluorescent signal to cross the threshold
- can be directly correlated to the RNA conc. of the sample

→ The lower the RNA concentration of the sample, the higher the Ct

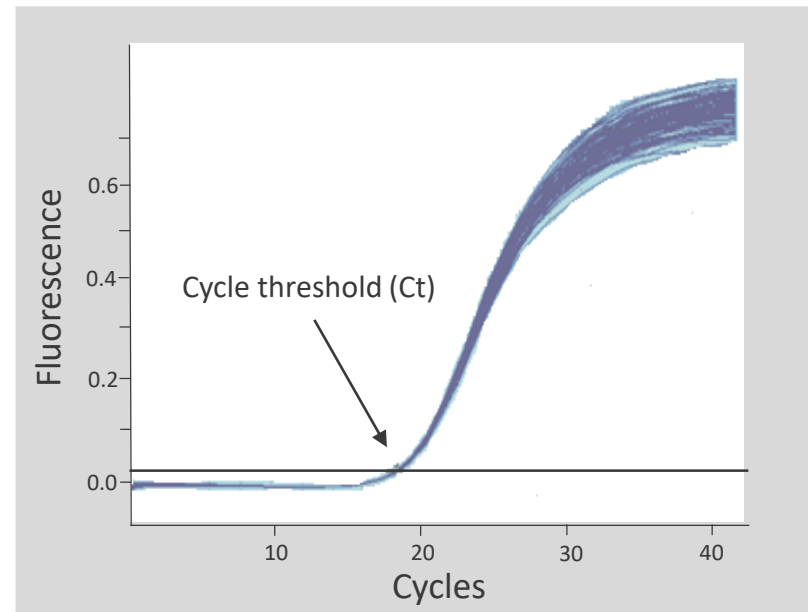

Benefits of using the NADAL® COVID-19 Ag Test

Test method vs. progression of infection*

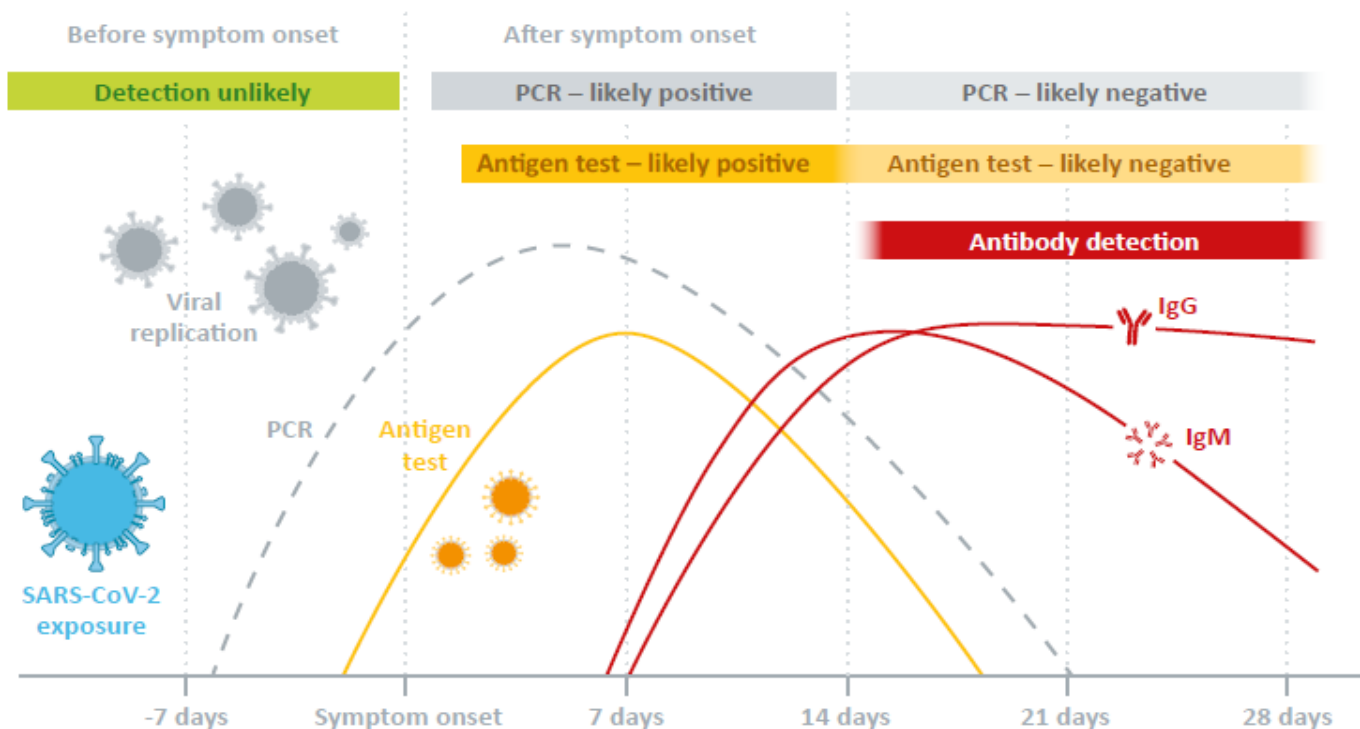

- ✓ Closing a diagnostic gap with antigen testing
- ✓ Similar detection windows for Ag detection & PCR

Antigen rapid tests currently on the market

- ✓ **Roche** SARS-CoV-2 Rapid Antigen Test (SD Biosensor)
 - Sensitivity: 96.52 % / specificity: 99.86 %
 - Literature results: Khairat et al. (2020)
 - [Instructions for use](#)
- ✓ **Abbott** Panbio COVID-19 Ag Rapid Test Device
 - Sensitivity: 93.3 % / specificity: 99.4 %
 - [Instructions for use](#)
- ✓ **Rapigen** Biocredit Covid-19 Ag
 - Sensitivity: 92 % / specificity: 98 %
 - Literature results: Khairat et al. (2020), Mak et al. (2020), Weitzel et al. (2020)
 - [Instructions for use](#)

Concluding facts

- ✓ **Applicable for all detection methods, incl. qRT-PCR:** A single negative test does not fully exclude the possibility of COVID-19 infection!
- ✓ **Combining forces is the key, as...**
 - ...“Each category of diagnostic test has its own unique role in the fight against this virus.”
 - U.S. Food and Drug Administration
 - NADAL[®] COVID-19 Ag Test benefits (slide [6](#))
 - NADAL[®] COVID-19 Ag Test **sensitivity of 97.56 %** at **Ct value of 20-30** (slide [8](#))
 - Specific marker **nucleocapsid protein** (slide [2](#))

Provide your customers with information

Please contact your service consultant. The following material is available:

- ✓ Landing page (DE, EN, ES, FR, CZ, FI)

www.nadal-test.com

- ✓ 243103N-20 product info flyer
- ✓ COVID-19 patient flyer
- ✓ Package insert

Altin Mehillka
Director Project Manager Sales

infin - Ingenieurgesellschaft für Informationstechnologien mbH
Waldbahnstr. 70, D-83324 Ruhpolding, Germany

Fon: +49 (0) 8663 7719954

Fax: +49 (0) 8663 7719950

Mail: a.mehillka@infin.de

Web: [www.https://www.infin.de/medizin/](https://www.infin.de/medizin/)



Complementary products for COVID-19 diagnostics

- ✓ 3170224 AFIAS **Vitamin D** Test
- ✓ 4x00x-Y Reactif **Urinalysis**
- ✓ 351006X-Y **D-Dimer** Test
- ✓ 3170202 AFIAS **D-Dimer** Test
- ✓ 2530104 **LEPU** Infrared forehead thermometer
- ✓ 311801X-Y **CRP**/ 311804N-20 NADAL **CRP plus** Test
- ✓ 312023NBUL-20 NADAL® **CRP Quant** Test
- ✓ 243001/ 243003 NADAL® **COVID-19 IgM/IgG** Test

Literatur

Bullard J, Dust K, *et al.* Predicting infectious SARS-CoV-2 from diagnostic samples. *Clin Infect Dis.* 2020 May. doi: [10.1093/cid/ciaa638](https://doi.org/10.1093/cid/ciaa638)

Chau NVV, Thanh Lam V, Thanh Dung N, *et al.* The natural history and transmission potential of asymptomatic SARS-CoV-2 infection. *Clin Infect Dis.* 2020 Jun 4;ciaa711. doi: 10.1093/cid/ciaa711. Epub ahead of print. PMID: 32497212; PMCID: PMC7314145.

Khairat SM, Guindy NEL *et al.* Evaluation of Two Rapid Antigen Tests for Detection of SARS-CoV-2. *J Microbiol Biotechnol*, 2020 Aug, doi: 10.11648/j.ijmb.20200503.18

Mak GCK, Cheng PKC *et al.* Evaluation of rapid antigen test for detection of SARS-CoV-2 virus. *J Clin Virol*, 2020 Aug, <https://doi.org/10.1016/j.jcv.2020.104500>

van Kampen JJA, van de Vijver DAMC, *et al.* Shedding of infectious virus in hospitalized patients with coronavirus disease-2019 (COVID-19): duration and key determinants. *Medrxiv.* 2020 Jun 9. doi: <https://doi.org/10.1101/2020.06.08.20125310>

Weitzel T, Legarraga P *et al.* Head-to-head comparison of four antigen-based rapid detection tests for the diagnosis of SARS-CoV-2 in respiratory samples. (preprint), *bioRxiv*, 2020 May, <https://doi.org/10.1101/2020.05.27.119255>

Woelfel R, Corman VM, *et al.* Virological assessment of hospitalized patients with COVID-2019. *Nature* 2020 Apr. <https://doi.org/10.1038/s41586-020-2196-x>