enverque

COVID-19 Antigen Rapid Test Kit

SARS-CoV-2-Antigen-Rapid Test Strips (Colloidal Gold-Immunochromatography)

- Result within 15 minutes!
- Teststrips: without any equipment and no need for pipettes
- CE mark and listed with the German Federal Institute for Pharmaceuticals and Medical (AT099/20)

Sensitivity :	98,5% (95%CI*: 97,20-99,80%)
Specificity :	99,3% (95%CI*: 98,50-100%)
Coincidence :	99,0% (95%CI*: 98,30-99,70%)

* Confidence-intervall%

	Procedure	Devices
1	Preparation of solution buffer (0,5ml = 10 drops)	Reaction tube
2	Collection of nasopharyngeal specimen and transfer into solution bugger	Nasopharynx swab
3	Insertion of the test strip into the prepared solution buffer	COVID-19 Antigen Test Strip
4	Reading of the test results within 15 minutes	(No need to remove the strip from tube)

Please read the instruction for use carefully!



Scope of delivery (one Covid-19 Antigen Rapid Test Kit): 50 single test strips in sealed packages

2 x 15ml solution buffer 1 x instruction for use

50 Reaction tubes50 single packed nasopharynx swabs

Packaging: 10 x 50 Test Kits (500 tests) in one standard shipping carton

COVID-19 Antigen Rapid Tests are for use by professional medical staff only

enverque GmbH Wollankstr. 131a 13187 Berlin, Germany GF: Joscha Hüdepohl Tel +49 30 3977 2992 backoffice@enverque.de www.enverque.de www.savena.info GLS Gemeinschaftsbank eG IBAN DE19430609671166570200 BIC GENODEM1GLS Amtsgericht: Charlottenburg HRB 162187 B VAT ID: DE 296919508 Tax Nr. 37/490/22226